


Donning Method of N95 – Lambda-Line Series

Dear Customers:

The purpose of this letter is to advise you that Shigematsu Works Co., Ltd. would like to present a proper donning method for our N95 – Lambda-Line series respirator. For the customers with smaller faces who cannot secure a proper fit around the chin, placing the lower strap over the crown of the head can improve the fit around the chin. This method of donning has been described in the user instructions.

This method of donning, however, was pointed out as inappropriate by NIOSH (National Institute for Occupational Safety and Health in the United States), so we have removed the relevant section from our User Instructions.

If it does not fit unless the lower strap is placed over the crown of the head, we kindly ask you to discontinue use.

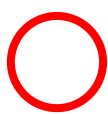

Correct Donning



Inappropriate Donning

The affected N95 Lambda-Line products are as follows.

1. DD02-N95+S
2. DD02-N95-2K
3. DD02V-N95-2K

If you have any questions or concerns, we kindly ask you to contact us at phone number below.

TEL: +81-3-6903-7528

3/22/2024

Masanori Nito

Senior Managing Director

Sales Division Manager & Overseas Division Manager

Shigematsu Works Co., Ltd.

1-26-1, Nishigahara, Kita-ku

Tokyo Japan 114-0024